**geemarc™** Hear the Difference **iLoop™+**

**Neck Loop  
Boucle Magnétique  
Induktionsschleife**

**English P1**

**Français P8**

**Deutsch P15**

## 1. GENERAL

---

The mobile iLoop™ + inductive loopset is for people with hearing aids.

When you use the iLoop™ +, you can speak directly to the caller by the microphone inside the system. So, the device works as a wireless system. Compatible with products equipped with a standard 3.5mm coaxial microphone, headset socket & with most cellular like iPhone, Blackberry, Samsung etc, and DECT....in some instances you may need to purchase an adaptor in case the product does not have a suitable headset socket.

***NOTE:*** Don't forget to switch your hearing aid to T mode.

**NOTE:**

- **OFF:** LED is off
- **ON:** LED is Green
- **Low battery:** LED is Red
- **Battery Charging:** LED is Blue

## 2. POWER SUPPLY / CONNECTING

The unit needs 2pcs AAA size 1.2V rechargeable NiMH batteries for correct functioning. The batteries must be charged for about 7 hours before use.

- Open the battery compartment door.
- Insert the batteries supplied with the unit, making sure the polarity is according to the diagram inside the battery compartment.
- To charge the batteries connect the supplied USB cable to the USB socket of the unit and then onto a USB power supply. The blue LED will light up.
- Continue charging until the LED switches OFF.
- Disconnect the USB cable after the charging is complete.
- Connect the audio plug of the unit to the microphone/headphone socket of your telephone. Your iLoop™ + is now ready for use.

**NOTE:**

- *Always disconnect the plug of the iLoop™+ from the telephone before installing the batteries.*
- *Check that the microphone/headphone socket is compatible with the 3 contact plug (speaker, mic, common).*
- *Do not dispose off batteries in household rubbish. Pay attention to the battery regulations regarding battery disposal.*

## 3. MAKING / RECEIVING CALLS

---

- Place the loop around your neck.
- Switch your hearing aid to **T** mode.
- Press the **ON/OFF** button. You are ready to speak through your iLoop<sup>TM</sup>+
- The Green LED will turn ON.
- During the call you can adjust the volume of your device by using the volume control knob located at the bottom of the iLoop<sup>TM</sup>+

The iLoop<sup>TM</sup>+ will be automatically turned OFF at the end of the call or after a few minutes of silence. It will also turn ON automatically when there is an Incoming call or if the person at the other end speaks after a long pause.

**NOTE:** Voltages presented to the Telecommunications network are classified TNV-3 according to EN60950 standard.

**NOTE:** When the batteries are discharged, the Red LED will light up. Charge the batteries immediately to avoid damage to the unit or the batteries.

## 4. GUARANTEE

---

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at [www.geemarc.com](http://www.geemarc.com). The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**The guarantee applies to the  
United Kingdom only.**

**DECLARATION** : Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)*

## 5. RECYCLING DIRECTIVES

---

**Recycling directives:** The WEEE (Waste Electrical and Electronic Equipment) Directive has been put in place so that products at the end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website  
at [www.geemarc.com](http://www.geemarc.com)

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